

SONIC

Important Information about Sonic SoundClip-A and TV-A Adapter Wireless Accessories

A Firmware Update is available for your accessory. Firmware controls the functionality and behavior of the device. When a new firmware version is available, a Firmware Update should be performed whenever possible.

The SoundClip-A and TV-A Firmware Update is not mandatory, however it will provide both sound quality and general stability improvements for the user.

In addition to new accessories, previously purchased stock accessories and those currently in use can be updated as well.

HOW TO UPDATE THE ACCESSORY'S FIRMWARE

Performing a Firmware Update is simple and can be done at any time:

1. Open the Sonic Firmware Updater in the Windows Start Menu
2. Connect the accessory to a USB cable
3. Plug USB cable into Fitting PC
4. Click on specified device
5. Click update

Contact Sonic Wholesale and Equipment on **1800 639 263** for any questions you may have before updating the firmware.