

MB 11 BERAphone®  
The key features at a glance:

- Fast and automated ABR-screening
- Unique BERAphone® with integrated electrodes saves costs for disposables
- CE-Chirp® – Stimulus
- Automatic Impedance Check indicates impedance conditions
- Export patient and test data for tracking purposes
- Stimulation level of 35 dBnHL
- Battery-operated when used with mobile PC

If you are looking for more detailed information please visit the MAICO Website [www.maico.biz](http://www.maico.biz) or contact your local MAICO Distributor.



**MAICO Diagnostics GmbH**

Sickingenstr. 70-71  
10553 Berlin, Germany

Tel.: +49 30 / 70 71 46-50  
Fax: +49 30 / 70 71 46-99

E-mail: [sales@maico.biz](mailto:sales@maico.biz)  
Web: [www.maico.biz](http://www.maico.biz)



7/2017

# MB 11 BERAphone®

## AABR Baby Screener


So easy to use



Apply electrode gel at the three electrode sites on the baby's head



Place a drop of electrode gel on the integrated electrodes

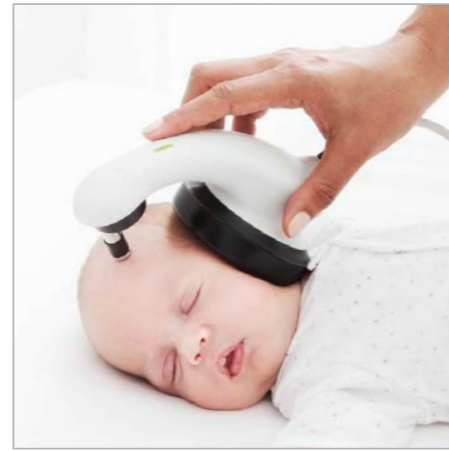

Position the MB 11 BERaphone®, surrounding the baby's ear with the earphone and resting the electrodes on the prepared sites

Baby-friendly –  
No adhesive disposables,  
no pulling, no pain.




User-Friendly Software



Start Screen – Simple user interface enhances ease of training



Traffic Light – “Traffic light” display for electrode contact and test quality provides easy to understand feedback to screeners.

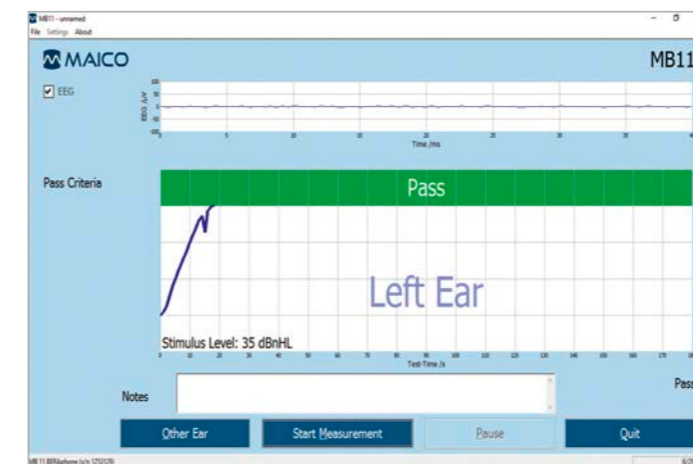

Pass Screen – Patented CE-Chirp® stimulus and advanced response detection algorithm allow fast test times.



Tracking – HiTrack and Oz compatible export functions available

State-of-the-Art AABR Screener

- Patented CE-Chirp® stimulus – achieves a larger ABR response and fast test times
- Fast stimulus rate and powerful response detection algorithm – processes more ABR response data in a shorter time period

